



HIV 1+2 / Syphilis Combo Card Test

Anti HIV 1+2 / Syphilis antibody detection card test
(Whole blood / Serum / Plasma)

FEATURES

- ▶ Clear background.
- ▶ Self contained test procedure.
- ▶ Specifically engineered for whole blood specimen, as well as serum and plasma.
- ▶ 100% sensitivity and 100% specificity.
- ▶ Clear differentiation between HIV & Syphilis infections.
- ▶ Detects antibodies to gp41 & gp36 of HIV.
- ▶ Detects antibodies to P47, P45, P17 & P15 of Syphilis.
- ▶ High-tech membrane strip locks in visible test results at ambient conditions even months later
- ▶ Storage temperature 4°C -30 °C & long shelf life up to 24 months.

BENEFITS

- ▶ Unmistakable results at 15 minutes.
- ▶ Simple and easy to use, does not require advanced medical training.
- ▶ Cost effective, can be used effectively in resource-limited conditions.
- ▶ Accurate and reliable.
- ▶ Helps to detect separately for HIV and Syphilis.
- ▶ Sensitive and accurate with most known subgroups and early sero conversion samples.
- ▶ Does not require cold chain & convenient for small volume users.


KIT CONTENTS

- Test device with desiccant
- Assay buffer bottle
- Specimen transfer device
- Sterile lancet
- Alcohol swab
- Instructions for use

EXTERNAL LABORATORY EVALUATION REPORTS


Place of Evaluation	Year of Testing	Sensitivity		Specificity	
		HIV	Syphilis	HIV	Syphilis
WHO Evaluation Report	2018	100%	99%	99.5%	100%
Ghana	2018	100%	100%	100%	100%
Zimbabwe	2019	100%	100%	100%	100%

TEST PROCEDURE

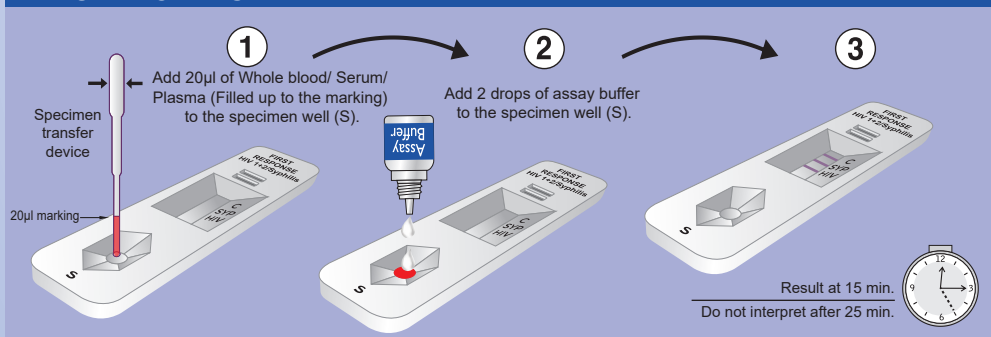


RESULTS INTERPRETATION

Negative HIV Positive Syphilis Positive HIV & Syphilis Positive

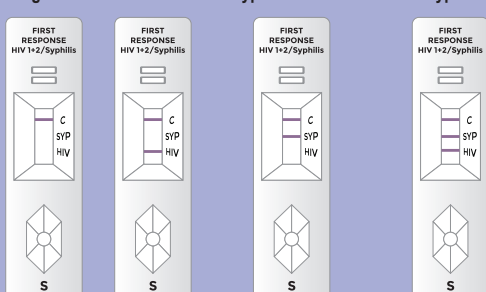

Invalid Results

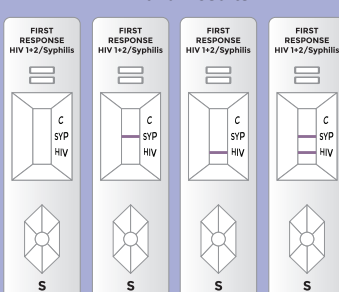

ORDERING INFORMATION

Info@premiermedcorp.com

Type : Bulk Packing

Catalogue No.	Pack Size
I20FRC25	25 Tests / Kit
I20FRC30	30 Tests / Kit
I20FRC50	50 Tests / Kit
I20FRC60	60 Tests / Kit
I20FRC100	100 Tests / Kit
I20FRC25-SA	25 Tests / Kit



ISO 13485 & EN ISO 13485
Certified Company

Manufactured by

Premier Medical Corporation Private Limited

Mfg. Unit : A1-302, GIDC, Sarigam 396155. Dist. Valsad, Gujarat, INDIA

• Customer support email : info@premiermedcorp.com • Tel.: +91 260 2780112/113,
• www.premiermedcorp.com