

1st FIRST RESPONSE[®]

HBsAg Card Test

For the detection of Hepatitis B surface antigen in human whole blood / serum / plasma



FIRST RESPONSE[®] HBsAg CARD TEST



Prepare all items shown above for testing. (Timer and gloves are not provided in the test kit; they must be available)




Confirm product name and check the sealing of test device pouch.



Check expiry date on the test pouch. (If expired, do not use)



Put on gloves. (Use new gloves for each client)

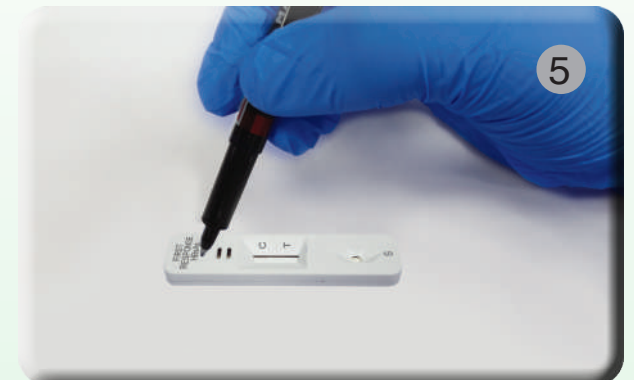

Write a client identifier on the test.




Wipe the client's finger with the alcohol swab. (Alcohol must be dry before pricking)



Remove clear protective cap of the lancet



Push lancet gently against test site. Wipe the first drop of blood and discard the lancet into sharp bin.

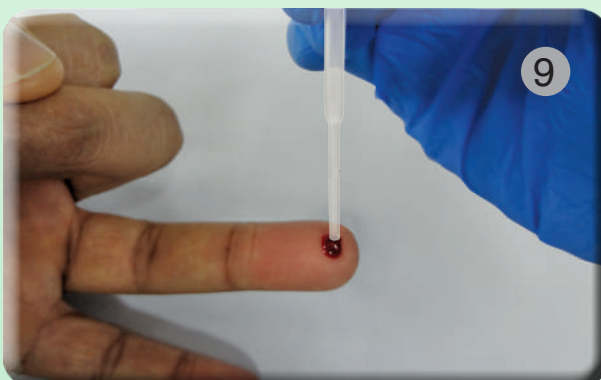

Gently squeeze the bulb of a specimen transfer device and touch the tip to a drop of blood. Release the bulb to draw up the blood to the specimen transfer device.



Add two drops of Whole blood/serum /plasma into the specimen well.



Add one drop of assay buffer into the specimen well.

INTERPRETATION



Read test results at 20 Minutes. Don't interpret after 20 Minutes.

